



Bacterial/Viral Filter

[CLICK TO LEARN MORE](#)

Warning: Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

Not made with natural rubber latex.



Warning: Be sure all connections are secure.

Bacterial Filtration Efficiency (BFE): 99.999+%

Viral Filtration Efficiency (VFE): 99.99+%

Dead Space: 42 mL

Flow Resistance: Approximately 1.5 cm/H₂O at 60 LPM

Connections: One (1) 22 mm I.D., one (1) 15 mm I.D./22 mm O.D.





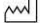




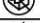
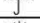


Single use: Do not reuse, reprocess or re-sterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.

Storage Instruction

Store in a cool and dry place, protected from light and ozone (no ultraviolet light, if applicable).

Keep away from sunlight and keep dry. Do not use if the packaging is compromised.

Explanation of important symbols and markings on the product label.

	Catalogue number
	Quantity, Number of items
	Batch code
	Date of manufacture
	Use by
	Product is non sterile
	Manufacturer
	Do not reuse
	Caution, Attention, Important, Note
	Consult instructions for use
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight
	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. (US Only)



Teleflex Medical Sdn. Bhd.
Lot PT 2577, Jalan Perusahaan 4,
34600 Kamunting, Perak, Malaysia

Hudson RCI and the Hudson RCI logo are registered trademarks of Teleflex Medical Incorporated in the United States of America and in selected other countries.

© 2020 Teleflex Medical Incorporated

International: (919) 544-8000

USA: (866) 246-6990

Teleflex®