# PARI LC<sup>®</sup> ADULT AEROSOL MASK

## INSTRUCTIONS FOR USE

For single patient use (one person) only. For use with PARI Nebulizers

[CLICK TO LEARN MORE](#)

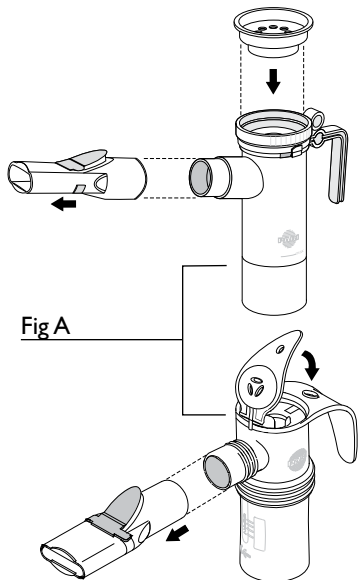
Fig A

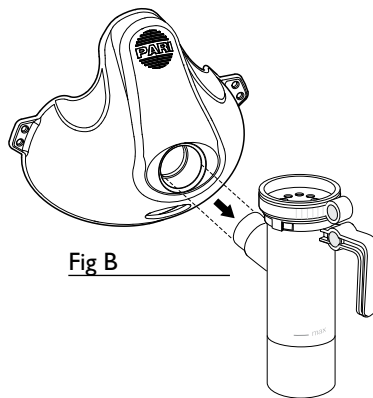
Fig B

### **⚠ WARNING**

READ ALL WARNINGS AND INSTRUCTIONS BEFORE USE. IMPROPER USE CAN RESULT IN SERIOUS OR FATAL ILLNESS OR INJURY.

PARI's LC<sup>®</sup> Adult aerosol mask is designed with Pro-Vent™ technology. This optimizes inhalation of respirable particles to the lungs while dramatically reducing facial & ocular deposition, resulting in safer treatments.

### **PLEASE FOLLOW STEP-BY-STEP AND REFER TO ILLUSTRATIONS ABOVE.**

1. **With PARI Nebulizer:** Remove the mouthpiece from the nebulizer and ensure inspiratory valve cap is attached (Fig A). Then press mask firmly onto the nebulizer outlet (Fig B).
2. **Elastic Strap:** Your mask includes an elastic strap which can be used for increased stability during treatments. If this is not needed, simply remove it from the mask and store in a safe place.
3. **Cleaning:** Wash with warm water and liquid dish soap. Rinse thoroughly then allow to dry on a clean, lint-free cloth and store. Do not place mask in dishwasher or autoclave.

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© 2014 PARI Respiratory Equipment, Inc.
**PARI Respiratory Equipment, Inc.**